




6104 Camino Verde Dr, #1
San Jose, CA 95119
(408)413-8593
sales@norcalcpap.com

Programming your unit remotely

Your prescribing physician can program your unit by connecting remotely via the integrated wireless module. To do this, simply provide the **SERIAL NUMBER (SN)** & **DEVICE NUMBER (DN)** in the back of your unit.

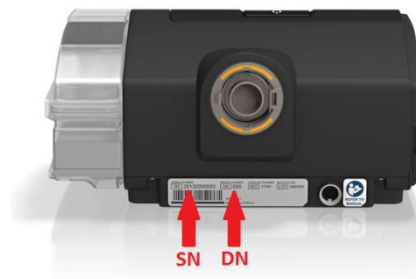

Programming your unit manually

To program your unit manually, you will need to access the clinical menu.

1. Press and hold the **DIAL** & **HOME** buttons simultaneously for 3 seconds.



2. Highlight "Settings" and press the **DIAL**.



3. Turn the **DIAL** and highlight the setting you want to adjust and press the **DIAL**.
4. Turn the **DIAL** to adjust the setting and press the **DIAL** to save the change.
5. Exit the clinical menu when finished by pressing & holding the **DIAL** & **HOME** buttons simultaneously for 3 seconds.